Instructions for use Contact™ Nipple Shield

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EN Contact™ Nipple Shield

Intended use
The Medela Contact Nipple Shields are intended to cover and protect the nipples of a mother to support breastfeeding.

Indications
The Contact Nipple Shields are indicated for use when the mother has flat or inverted nipples, sore nipples or suffers from nipple pain. They can also be used when the baby has latch-on problems (such as an atypical sucking technique), when the baby does not transfer the milk well or for preterm infants who fall asleep almost immediately at the breast.

Contraindications
There are no known contraindications.

Important safety information
Read all instructions before use
Use the product only for its intended use as described in this instruction manual.
Use by more than one person without adequate reprocessing may be a health risk and could cause cross-contamination.
Use Medela original accessories only.
Do not use if the packaging is open or damaged before first use.

Instructions before each use. Throw away at the first signs of damage or weakness.
Never leave packaging and components unattended. Keep them out of reach of children.
If rashes or pain occur, consult your lactation consultant or physician.
SAVE THIS INSTRUCTION MANUAL

Cleaning
Only use drinking-quality water for cleaning.
Process the shield and the storage box prior to first use.
Clean all parts immediately after use.
The cleaning instructions below are general instructions. For country-specific regulations and guidelines, ask your doctor, pharmacist or visit website (www.medela.com).

Before first use and once a day
Clean the shield and the storage box as described under "After each use."
Disinfect the shield and the storage box as described under "Disinfection."

After each use
Rinse the parts with cold water (approx. 20 °C). Clean the shield with plenty of warm, soapy water (approx. 30 °C). Use a commercially available washing-up liquid, preferably without artificial fragrances and colouring. Rinse parts with cold water.
Dry with a clean cloth or leave to dry on a clean cloth.
Alternatively, clean the parts in the dishwasher. Place them on the top rack or in the cutlery section. Use a commercially available dishwashing detergent.

Disinfection
Clean the product as described under "Cleaning."
Cover the parts with water and boil for five minutes.
Or use a Quick Clean microwave bag in accordance with the instructions.
Dry with a clean cloth or leave to dry on a clean cloth.

Storage
Put the clean nipple shield in the clean storage box, if available. Alternatively, put the shield in a clean storage bag or a clean environment. It is important that all residual moisture dries. Do not store in an airtight container/bag.

How to use
Wash your hands thoroughly with drinking-quality water and soap before touching the shield and the breast. Dry your hands with a fresh towel or a single-use paper towel.
Clean the breast and surrounding area with a damp cloth. Do not use soap or alcohol which can dry the skin and cause cracked nipples.
Choose the correct nipple shield size
When choosing the correct nipple shield size you need to consider both mother and baby.

Product variants
Available in 3 sizes: S/16mm, M/20mm or L/24mm.
Material: nipple shield: Silicone
Material storage box: Polypropylene (PP)
Disposal: According to local regulations

Compliance with the essential requirements of the Directive 93/42/EEC of 14 June 1993 concerning medical devices.

The packaging contains products intended to come in contact with food.
Do keep the device away from sunlight.

Do keep the device away from sunlight.

Do keep the device away from sunlight.

Do keep the device away from sunlight.

Do keep the device away from sunlight.

Do keep the device away from sunlight.

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