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TREATMENT OF EXUDATIVE CUTANEOUS LESIONS

The term exudative dermatitis refers to moist skin irritation which progresses to suppuration or maceration of variable origin and may frequently affect both adults and children of all ages.

A great number of conditions may be accompanied by skin lesions or moist skin irritation, such as atopy, allergies, skin infections, superficial skin ulcers, etc.

Acting externally through direct topical application, zinc, copper and aluminium-potassium sulphates present in **Septomida MD** exert drying and astringent effects, eliminating exudate from exudative cutaneous lesions. This anti-exudative effect reduces the risk of infections.

Septomida MD is indicated as an antiexudative, astringent and antiseptic co-adjutant for weeping skin lesions in adults and children.

Septomida MD may be used in combination with medicated treatments.

INDICATIONS

Cleaning wounds, hygiene of weeping skin ulcers (pressure sores, diabetic ulcers, burns, etc.), or any other weeping lesion that is infected or likely to become infected. It is used as a co-adjutant treatment for weeping lesions of allergic (acute dermatitis, acute exudative eczema), fungal (exudative dermatomycosis, infections with vesicles/macération, etc.), bacterial (acute secondary dermatitis, pompholyx, impetigo) and viral aetiology (herpes zoster), as well as inflammatory skin lesions with exudate (diaper rash, intertrigo, non-specific or irritative external exudative balanitis or balanoposthitis, etc.).

INSTRUCTIONS FOR USE

1) **How to prepare the solution:** Fill a suitable container with tap water. Add the contents of approximately one sachet per litre and stir until the powder is fully dissolved. The solution is now ready for use. 2) **Application:** Bathe the affected area in the sink, bidet, a wash bowl or suitable container twice daily until the exudate or moisture disappears.

COMPOSITION

Main ingredients: zinc sulphate (22,23%), aluminium-potassium sulphate (22,23%) and copper sulphate (11%).

WARNINGS

Carefully follow the directions for use. Avoid contact with eyes. In case of accidental contact, rinse with plenty of water. Do not ingest. For external use only. If lesions show no sign of improvement after 15 days of treatment, seek medical advice. Do not use after the expiry date indicated on the package. The validity period refers to the intact product which has been properly stored. Keep out of the sight and reach of children. Dispose of the product appropriately after use. Inform the manufacturer and competent authority of any severe adverse events associated with the product.

PRESENTATION

Pack with 12 single-dose sachets containing 9 g of powder.

Keep these instructions for use until the end of the product's useful life.

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